

Assay ID	Assay Type	False Biotin Effect	Biotin Threshold ng/mL
ACOV2	Sandwich	Negative	1200
ACOV2S	Sandwich	Decrease	1200
AHAV 2	Competitive	Positive	100
AFP	Sandwich	Decrease	60
B12 2	Competitive	Increase	50
CA 125 2	Sandwich	Decrease	35
CA 19-9	Sandwich	Decrease	30
CEA	Sandwich	Decrease	70
CORT 2	Competitive	Increase	30
CPEPTID	Sandwich	Decrease	60
E2 3	Competitive	Increase	36
FERR	Sandwich	Decrease	50
FOL	Competitive	Increase	21
FPSA	Sandwich	Decrease	100
FSH	Sandwich	Decrease	60
FT3 3	Competitive	Increase	70
FT4 2	Competitive	Increase	20
HCG-BETA	Sandwich	Decrease	70
INSULIN	Sandwich	Decrease	60
LH	Sandwich	Decrease	50
PRL 2	Sandwich	Decrease	40
PROG 3	Competitive	Increase	30
PTH	Sandwich	Decrease	50
RUBIGG	Sandwich	Non-Reactive	50
T3	Competitive	Increase	10
T4	Competitive	Increase	100
TESTO 2	Competitive	Increase	30
TPSA	Sandwich	Decrease	80
TSH	Sandwich	Decrease	1200
T UP	Competitive	Increase	40
VITDT 2	Competitive	Increase	30